



eng



# DMV Magic Touch™

## For Hard & Gas Permeable Contact Lenses

### Intended Use

This device is designed for inserting and removing hard and gas permeable contact lenses.

### Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

**REMOVAL** – Moisten the suction cup. Firmly squeeze the middle of the handle. Be sure the contact lens is centered on the eye. Align the suction cup in front of the lens and touch it to the center of the lens by using a mirror. Keep a tight grip on the handle to create suction. Take the contact lens directly from the eye. After removal, release the tight grip on the handle to slide the lens off the suction cup.

**INSERTION** – Place the wet contact lens on the suction cup. The wet lens will adhere in this position. **DO NOT** squeeze the handle. Place the contact lens directly on the eye.

Follow the directions of your contact lens fitter / eye care professional.

### Cleaning

Each time after using the DMV Magic Touch clean it with 70% isopropyl alcohol and allow to air dry. Store the device in its case after dry. **Keep the air channel free of debris.** Plan to replace it every six months.



**Precaution:** If the DMV Magic Touch would accidentally become attached to the eye, release the handle and it will fall from the eye. This device will not stick to the eye.



**Caution:** Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



**Caution: Choking hazard.** Keep this and all other medical devices away from small children.

This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



**Caution:** Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex,** a known allergen.



**Do NOT use DMV Magic Touch for SOFT or Hybrid contact lenses**

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.

### Waste Disposal

DMV Corporation recommends to dispose used DMV products according to local regulations in force at the moment of disposal.



This product is a medical device



Protect from direct Sun



Importer



Do not store warmer than 105°C/220°F



Single patient - multiple use



Manufacturer



Read Instructions for Use [dmvcorp.com](http://dmvcorp.com)



Date of Manufacturing



Country of manufacture



Authorized Representative



EU  
Bilosa Handels GmbH  
Haller Strasse 109  
6020 Innsbruck, Austria



Emergo Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands



Switzerland  
Prolens AG, Josefstrasse 53  
CH-8005, Zürich, Switzerland



MedEnvoy Switzerland  
Gotthardstrasse 28  
CH-6302 Zug, Switzerland



DMV Corporation  
1024 Military Road  
Zanesville, Ohio 43701 - USA



UK  
IGO Optical Ltd  
The Granary, Manor Park, Warkworth  
Banbury OX17 2AG, UK



Emergo Consulting (UK)  
Limited c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge CB24 9BZ, UK

