




eng

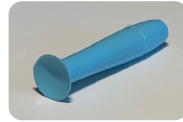

DMV Ultra™ / DMV 45™

For Hard & Gas Permeable Contact Lenses

Intended Use

DMV Ultra and DMV 45 are both designed for inserting and removing Hard and Gas Permeable contact lenses. The DMV Ultra should be held vertically and the DMV 45 held at an angle.

Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

REMOVAL – Moisten the suction cup of the DMV Ultra/DMV 45 and guide it to the eye. Be sure the contact lens is centered on the eye. Align the suction cup in front of the lens and touch it to the center of the lens by using a mirror. Take the contact lens directly from the eye. After removal, carefully slide the contact lens sideways from the suction cup.

INSERTION – Place the wet contact lens on the shallow dimple in the narrow end of the handle. The wet lens will adhere in this position. Touch the lens to the eye.

Follow the directions of your contact lens fitter / eye care professional.

Cleaning

Each time after using the DMV Ultra /DMV 45 clean it with 70% isopropyl alcohol and allow to air dry. Store the device in its case after drying. Plan to replace it every six months.



Precaution: If the device would accidentally become attached to the eye, slide it to the corner of the eye and remove it by turning.



Caution: Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



Caution: Choking hazard. Keep this and all other medical devices away from small children.

This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



Caution: Though there is no evidence of allergy issues with this product, it **does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



Do NOT use DMV Classic for SOFT or Hybrid contact lenses

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.

Waste Disposal

DMV Corporation recommends to dispose used DMV products according to local regulations in force at the moment of disposal.



This product is a medical device



Protect from direct Sun



Importer



Do not store warmer than 105°C/220°F



Single patient - multiple use



Manufacturer



Read Instructions for Use dmvcorp.com



Date of Manufacturing



Country of manufacture



Authorized Representative



EU
Bilosa Handels GmbH
Haller Strasse 109
6020 Innsbruck, Austria



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



Switzerland
Prolens AG, Josefstrasse 53
CH-8005, Zürich, Switzerland



MedEnvoy Switzerland
Gotthardstrasse 28
CH-6302 Zug, Switzerland



UK
IGO Optical Ltd
The Granary, Manor Park, Warkworth
Banbury OX17 2AG, UK



Emergo Consulting (UK)
Limited c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ, UK



DMV Corporation
1024 Military Road
Zanesville, Ohio 43701 - USA

