



eng



# DMV Luma Serter Plus™

## Luminous Inserter for large lenses

### Intended Use

DMV Luma-Serter Plus is designed for inserting contact lenses with a larger diameter than 13 mm. It can be used for Scleral, Soft, Hybrid or Gas Permeable (firm) contact lenses.

### Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

**INSERTION** – Put the device on the lighted end of a penlight. Place the contact lens on the centre of the cup. Follow the lens preparation instructions provided by your eye care practitioner. Turn on the penlight and gently move the lens toward the open eye by using the illuminated centre of the cup as a guide. Hold the eye lids open with the other hand and place the lens on the cornea. Once the lens has adhered to the eye, withdraw the Luma-Serter Plus. The light filter prevents contact lens solutions from passing through to the penlight. The device is contoured to fit the end of most penlights.

If patients prefer, they may choose to place the device on the end of a finger instead of a penlight. The ribs in the cup are designed to provide an elevated surface for the lens to rest on and prevent the lens from sticking to the Luma-Serter Plus.

Follow the directions of your contact lens fitter / eye care professional.

### Cleaning

Clean the device after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case after dry. Plan to replace it every six months.



**Precaution:** If you apply the device accidentally to the surface of the eye, simply remove it from the eye, as there is no suction. Avoid sliding the device over the surface of the eye.



**Caution:** Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



**Caution: Choking hazard.** Keep this and all other medical devices away from small children.

This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



**Caution:** Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex, a known allergen.**



**DO NOT USE Luma-Serter Plus with contact lenses smaller than 12.5 mm, to avoid touching the surface of the eye.**

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.

### Waste Disposal

DMV Corporation recommends to dispose used DMV products according to local regulations in force at the moment of disposal.



This product is a medical device



Protect from direct Sun



Importer



Do not store warmer than 105°C/220°F



Single patient - multiple use



Manufacturer



Read Instructions for Use [dmvcorp.com](http://dmvcorp.com)



Date of Manufacturing



Country of manufacture



Authorized Representative



EU  
Bilosa Handels GmbH  
Haller Strasse 109  
6020 Innsbruck, Austria



Emergo Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands



Switzerland  
Prolens AG, Josefstrasse 53  
CH-8005, Zürich, Switzerland



MedEnvoy Switzerland  
Gotthardstrasse 28  
CH-6302 Zug, Switzerland



UK  
IGO Optical Ltd  
The Granary, Manor Park, Warkworth  
Banbury OX17 2AG, UK



Emergo Consulting (UK)  
Limited c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge CB24 9BZ, UK



DMV Corporation  
1024 Military Road  
Zanesville, Ohio 43701 - USA

