



eng



DMV Scleral Cup™

For Scleral Contact Lenses and Prosthetic Eyes

Intended Use

DMV Scleral Cup is designed for inserting/removing Scleral contact lenses and removing Prosthetic Eyes.

Instructions for Use

Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

REMOVAL – Moisten the suction cup. Firmly squeeze the middle of the handle. Make sure the contact lens is centred on tMoisten the suction cup and squeeze the handle. Looking in the mirror, place the suction cup against the scleral contact lens. Release the tension on the handle to allow suction to hold the contact lens. The contact lens can then be manipulated in such a way as to allow the contact lens to be removed from the eye. Follow the directions of your contact lens fitter for the best technique. After removal, squeeze the handle again to release the suction and remove the lens from the suction cup..

INSERTION – Place the wet contact lens on the cup without compressing the handle. The wet lens will adhere to the cup. **DO NOT** squeeze the handle while inserting the lens. Gently place the scleral lens in the open eye.

Follow the directions of your contact lens fitter / eye care professional.

Cleaning

Clean the DMV Scleral Cup after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case once dry. **Keep the air channel free of debris.** Plan to replace it every six months.

Precaution: If the DMV Scleral Cup accidentally becomes attached to the eye, squeeze the handle slightly to release the suction and remove it from the eye.

Caution: Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.

Caution: Choking hazard. Keep this and all other medical devices away from small children.

This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.

Caution: Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex,** a known allergen.

Do NOT use DMV Scleral Cup for SOFT or Hybrid contact lenses.

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.

Waste Disposal

DMV Corporation recommends to dispose used DMV products according to local regulations in force at the moment of disposal.



This product is a medical device



Protect from direct Sun



Importer



Do not store warmer than 105°C/220°F



Single patient - multiple use



Manufacturer



Read Instructions for Use dmvcorp.com



Date of Manufacturing



Country of manufacture



Authorized Representative



EU
Bilosa Handels GmbH
Haller Strasse 109
6020 Innsbruck, Austria



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



Switzerland
Prolens AG, Josefstrasse 53
CH-8005, Zürich, Switzerland



MedEnvoy Switzerland
Gotthardstrasse 28
CH-6302 Zug, Switzerland



UK
IGO Optical Ltd
The Granary, Manor Park, Warkworth
Banbury OX17 2AG, UK



Emergo Consulting (UK)
Limited c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ, UK



DMV Corporation
1024 Military Road
Zanesville, Ohio 43701 - USA

