




eng

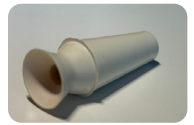

DMV Vented Scleral Cup™

For Hybrid and Scleral Contact Lenses

Intended Use

DMV Vented Scleral Cup is designed for inserting Hybrid and Scleral contact lenses.

Instructions for Use

Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

INSERTION – An air channel runs through the device to disable suction. This allows a contact lens to be placed on the eye without sticking to the device. Hold the device in a vertical position and place the contact lens in the suction cup. Make sure to fill the lens with approved solution when inserting Hybrid or Scleral lenses. Looking in the mirror, carefully raise the device to the open eye and touch the lens to the eye. Follow the directions of your contact lens fitter / eye care professional.

Cleaning

Clean the DMV Vented Scleral Cup after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case once dry. **Keep the air channel free of debris.** Plan to replace it every six months.

Cleaning

Clean the DMV Scleral Cup after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case once dry. **Keep the air channel free of debris.** Plan to replace it every six months.

Precaution: If you apply the device accidentally to the surface of the eye, simply remove it from the eye, as there is no suction. Avoid sliding the device over the surface of the eye.

Caution: Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.

Caution: Choking hazard. Keep this and all other medical devices away from small children. This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.

Caution: Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex,** a known allergen.

DO NOT USE this device FOR REMOVAL of contact lenses.

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.

Waste Disposal

DMV Corporation recommends to dispose used DMV products according to local regulations in force at the moment of disposal.



This product is a medical device



Protect from direct Sun



Importer



Do not store warmer than 105°C/220°F



Single patient - multiple use



Manufacturer



Read Instructions for Use dmvcorp.com



Date of Manufacturing



Country of manufacture



Authorized Representative



EU
Bilosa Handels GmbH
Haller Strasse 109
6020 Innsbruck, Austria



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



Switzerland
Prolens AG, Josefstrasse 53
CH-8005, Zürich, Switzerland



MedEnvoy Switzerland
Gotthardstrasse 28
CH-6302 Zug, Switzerland



UK
IGO Optical Ltd
The Granary, Manor Park, Warkworth
Banbury OX17 2AG, UK



Emergo Consulting (UK)
Limited c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ, UK



DMV Corporation
1024 Military Road
Zanesville, Ohio 43701 - USA

