



eng

## DMV Versa™

### Remover/Inserter for Scleral Contact Lenses

#### Intended Use

DMV Versa is designed for inserting and removing scleral contact lenses.

#### Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

**REMOVAL** – Moisten the suction cup (smaller end) of the DMV Versa with saline solution, guide it to the eye and place it onto the front of the scleral contact lens near the edge. Slowly pull the lens away from the eye. This action will loosen the adherence of the lens and the lens will come free from the eye. After removal, carefully slide the contact lens sideways from the suction cup

**INSERTION** – Place a wet scleral lens onto the large end of the Versa where the lens will lightly adhere. Hold the eye lids open with the other hand and place the scleral contact lens on the eye. Once the lens has adhered to the eye, withdraw the DMV Versa. The ribs in the cup are designed to provide an elevated surface for the lens to rest on and prevent the lens from sticking to the Versa. Use a mirror for guidance. Follow the directions of your contact lens fitter / eye care professional.

#### Cleaning

Each time after using the device clean it with 70% isopropyl alcohol and allow to air dry. Store the device in its case after dry. Plan to replace it every six months.



**Precaution:** If the DMV Versa would accidentally become attached to the eye with the small end (suction cup), slide it to the corner of the eye and remove it by turning. If the DMV Versa would accidentally become attached to the eye with the large end (resting pad), merely withdraw it from the eye, as there is no suction on this end..



**Caution:** Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



**Caution: Choking hazard.** Keep this and all other medical devices away from small children.

This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



**Caution:** Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



**Do NOT use DMV Versa for removing SOFT or Hybrid contact lenses.**

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.

#### Waste Disposal

DMV Corporation recommends to dispose used DMV products according to local regulations in force at the moment of disposal.



This product is a medical device



Protect from direct Sun



Importer



Do not store warmer than 105°C/220°F



Single patient - multiple use



Manufacturer



Read Instructions for Use  
dmvcorp.com



Date of Manufacturing



Country of manufacture



Authorized Representative



EU  
Bilosa Handels GmbH  
Haller Strasse 109  
6020 Innsbruck, Austria



Emergo Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands



Switzerland  
Prolens AG, Josefstrasse 53  
CH-8005, Zürich, Switzerland



MedEnvoy Switzerland  
Gotthardstrasse 28  
CH-6302 Zug, Switzerland



UK  
IGO Optical Ltd  
The Granary, Manor Park, Warkworth  
Banbury OX17 2AG, UK



Emergo Consulting (UK)  
Limited c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge CB24 9BZ, UK



DMV Corporation  
1024 Military Road  
Zanesville, Ohio 43701 - USA

