



## DMV Classic™

### For Hard & Gas Permeable Contact Lenses

#### Intended Use

DMV Classic is designed for inserting and removing Hard and Gas Permeable contact lenses.

#### Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

**REMOVAL** – Moisten the suction cup. Be sure the contact lens is centred on the eye. Compress the handle of the Classic and guide it to the centre of the contact lens. Looking in the mirror, touch it to the front of the lens. While the Classic is touching the lens, slightly release the pressure on the handle to cause the Classic to adhere to the lens. Take the contact lens directly from the eye. After removal, squeeze the handle again to release the suction and remove the lens from the suction cup

**INSERTION** – Place the wet contact lens on the cup without compressing the handle. The wet lens will adhere to the cup. **DO NOT** squeeze the handle while inserting the lens. Place the contact lens directly on the eye.

Follow the directions of your contact lens fitter / eye care professional.

#### Cleaning

Clean the DMV Classic after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case once dry. **Keep the air channel free of debris.** Plan to replace it every six months.



**Precaution:** If the DMV Classic accidentally becomes attached to the eye, squeeze the handle to release the suction and remove it from the eye.



**Caution:** Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



**Caution: Choking hazard.** Keep this and all other medical devices away from small children. This product is not recommended for use with diseased or injured eyes. Consult your eye care professional if you experience eye irritation, unusual visual impairment or pain.



**Caution:** Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



**Do NOT use DMV Classic for SOFT or Hybrid contact lenses**

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State where you live.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.



This product is a medical device



Protect from direct Sun



Do not store warmer than 105°C/220°F



Single patient - multiple use



Read Instructions for Use  
dmvcorp.com



Date of Manufacturing



Distribution Europe:  
Prolens AG, Josefstrasse 53  
CH-8005, Zürich



DMV Corporation  
1024 Military Road  
Zanesville, Ohio 43701 - USA



Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague; The Netherlands