



DMV Luma Sertter Plus™

Luminous Inserter for large Lenses

DMV Luma-Sertter Plus is designed for inserting contact lenses with a larger diameter than 13mm. It can be used for Scleral, Soft, Hybrid or Gas Permeable (firm) contact lenses.

Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

INSERTION – Put the device on the lighted end of a penlight. Place the contact lens on the center of the cup.

Follow the lens preparation instructions provided by your eye-care practitioner. Turn on the penlight and gently move the lens toward the open eye by using the illuminated center of the cup as a guide. Hold the eye lids open with the other hand and place the lens on the cornea. Once the lens has adhered to the eye, withdraw the Luma-Sertter Plus. The light filter prevents contact lens solutions from passing through to the penlight. The device is contoured to fit the end of most penlights.

If patients prefer, they may choose to place the device on the end of a finger instead of a penlight.

The ribs in the cup are designed to provide an elevated surface for the lens to rest on and prevent the lens from sticking to the Luma-Sertter Plus.

Follow the directions of your contact lens fitter / eye care professional.

Cleaning

Each time after using the device clean it with 70% isopropyl alcohol and allow to air dry. Store the device in its case after dry. Plan to replace it every six months.



Precaution: If you apply the device accidentally to the surface of the eye, merely withdraw it from the eye, as there is no suction. Avoid sliding the device over the surface of the eye.



Caution: Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



Caution: Choking hazard. Keep this and all other medical devices away from small children. This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



Caution: Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



DO NOT USE Luma-Sertter Plus with contact lenses smaller than 12.5mm, to avoid touching the surface of the eye.

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.



This product is a medical device



Protect from direct Sun



Do not store warmer than 105°C/220°F



Single patient - multiple use



Read Instructions for Use dmvcorp.com



Date of Manufacturing



Distribution Europe:
Prolens AG, Josefstrasse 53
CH-8005, Zürich



DMV Corporation
1024 Military Road
Zanesville, Ohio 43701 - USA



Emergo Europe
Prinsessegracht 20
2514 AP, The Hague; The Netherlands