



# DMV Soft Lens Handler™

## Remover/Inserter for Soft Contact Lenses

### Intended Use

DMV Soft Lens handler is designed for inserting and removing soft contact lenses.

### Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

**REMOVAL** – Press the gripper pads firmly against the front of the soft contact lens. Look in the mirror to make sure that the lens is on the cornea. Gently squeeze the grippers together to extract the contact lens.

**INSERTION** – Place a wet contact lens on the small end of the device where the arrow indicates. Allow the lens to overlap the end of the device so you can view your progress through the lens.

Follow the directions of your contact lens fitter / eye care professional.

### Cleaning

Clean the device after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case after dry. Plan to replace it every six months.



**Precaution:** If you apply the device accidentally to the surface of the eye, simply remove it from the eye, as there is no suction. Avoid sliding the device over the surface of the eye.



**Caution:** Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



**Caution: Choking hazard.** Keep this and all other medical devices away from small children. This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



**Caution:** Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



**DO NOT USE** this device with **hard, gas permeable, scleral or hybrid contact lenses.**

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.



This product is a medical device



Protect from direct Sun



Do not store warmer than  
105°C/220°F



Single patient - multiple use



Read Instructions for Use  
dmvcorp.com



Date of Manufacturing



Distribution Europe:  
Prolens AG, Josefstrasse 53  
CH-8005, Zürich



DMV Corporation  
1024 Military Road  
Zanesville, Ohio 43701 - USA



Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague; The Netherlands