



# DMV Magic Touch™

## For Hard & Gas Permeable Contact Lenses

### Intended Use

This device is designed for inserting and removing hard and gas permeable contact lenses.

### Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

**REMOVAL** – Moisten the suction cup. Firmly squeeze the middle of the handle. Make sure the contact lens is centred on the eye. Looking in the mirror, align the suction cup in front of the lens and touch it to the centre of the lens. Keep a tight grip on the handle to create suction. Take the contact lens directly from the eye. After removal, release the tight grip on the handle to slide the lens off the suction cup.

**INSERTION** – Place the wet contact lens on the suction cup. The wet lens will adhere in this position. DO NOT squeeze the handle. Place the contact lens directly on the eye.

Follow the directions of your contact lens fitter / eye care professional.

### Cleaning

Clean the DMV Magic Touch after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case once dry. **Keep the air channel free of debris.** Plan to replace it every six months.



**Precaution:** If the DMV Magic Touch accidentally becomes attached to the eye, release the handle and it will fall from the eye. This device will not stick to the eye.



**Caution:** Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



**Caution: Choking hazard.** Keep this and all other medical devices away from small children. This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



**Caution:** Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



**Do NOT use DMV Magic Touch for SOFT or Hybrid contact lenses**

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.



This product is a medical device



Protect from direct Sun



Do not store warmer than 105°C/220°F



Single patient - multiple use



Read Instructions for Use  
dmvcorp.com



Date of Manufacturing



Distribution Europe:  
Prolens AG, Josefstrasse 53  
CH-8005, Zürich



DMV Corporation  
1024 Military Road  
Zanesville, Ohio 43701 - USA



Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague; The Netherlands