



DMV Ultra™ / DMV 45™

For Hard & Gas Permeable Contact Lenses

Intended Use

DMV Ultra and DMV 45 are both designed for inserting and removing Hard and Gas Permeable contact lenses. The DMV Ultra should be held vertically and the DMV 45 held at an angle.

Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

REMOVAL – Moisten the suction cup of the DMV Ultra / DMV 45 and guide it to the eye. Make sure the contact lens is centred on the eye. Looking in the mirror, align the suction cup in front of the lens and touch it to the centre of the lens. Take the contact lens directly from the eye. After removal, carefully slide the contact lens sideways from the suction cup.

INSERTION – Place the wet contact lens on the shallow dimple in the narrow end of the handle. The wet lens will adhere in this position. Touch the lens to the eye.

Follow the directions of your contact lens fitter / eye care professional.

Cleaning

Clean the DMV Ultra / DMV 45 after every use with 70% isopropyl alcohol and allow to air dry. Store the device in its case once dry. Plan to replace it every six months.



Precaution: If the device would accidentally become attached to the eye, slide it to the corner of the eye and remove it by turning.



Caution: Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



Caution: Choking hazard. Keep this and all other medical devices away from small children. This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



Caution: Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



Do NOT use DMV Ultra / DMV 45 for SOFT or Hybrid contact lenses

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.



This product is a medical device



Protect from direct Sun



Do not store warmer than 105°C/220°F



Single patient - multiple use



Read Instructions for Use
dmvcorp.com



Date of Manufacturing



Distribution Europe:
Prolens AG, Josefstrasse 53
CH-8005, Zürich



DMV Corporation
1024 Military Road
Zanesville, Ohio 43701 - USA



Emergo Europe
Prinsessegracht 20
2514 AP, The Hague; The Netherlands