



DMV Vented Scleral Cup™

For Hybrid and Scleral Contact Lenses

Intended Use

DMV Vented Scleral Cup is designed for inserting Hybrid and Scleral contact lenses.

Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

INSERTION – An air channel runs through the device to disable suction. This allows a contact lens to be placed on the eye without adhering to the device. Hold the device in a vertical position and place the contact lens in the suction cup. Be sure to fill the lens with approved solution when inserting Hybrid or Scleral lenses. Carefully raise the device to the open eye and touch the lens to the eye by using a mirror. Follow the directions of your contact lens fitter / eye care professional.

Cleaning

Each time after using the DMV Vented Scleral Cup clean it with 70% isopropyl alcohol and allow to air dry. Store the device in its case after dry. **Keep the air channel free of debris.** Plan to replace it every six months.



Precaution: If you apply the device accidentally to the surface of the eye, merely withdraw it from the eye, as there is no suction. Avoid sliding the device over the surface of the eye.



Caution: Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



Caution: Choking hazard. Keep this and all other medical devices away from small children. This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



Caution: Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex**, a known allergen.



DO NOT USE this device FOR REMOVAL of contact lenses.

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.



This product is a medical device



Protect from direct Sun



Do not store warmer than 105°C/220°F



Single patient - multiple use



Read Instructions for Use
dmvcorp.com



Date of Manufacturing



Distribution Europe:
Prolens AG, Josefstrasse 53
CH-8005, Zürich



DMV Corporation
1024 Military Road
Zanesville, Ohio 43701 - USA



Emergo Europe
Prinsessegracht 20
2514 AP, The Hague; The Netherlands