



eng



DMV Soft Lens Handler™

Remover / Inserter for Soft Contact Lenses

Intended Use

DMV Soft Lens handler is designed for inserting and removing soft contact lenses.

Instructions for Use



Before using the first time, clean the device with 70% isopropyl alcohol and allow to air dry.

REMOVAL – Press the gripper pads firmly against the front of the soft contact lens. Be sure that the lens is on the cornea by using a mirror. Gently squeeze the grippers together to extract the contact lens.

INSERTION – Place a wet contact lens on the small end of the device where the arrow indicates. Allow the lens to overlap the end of the device so you can view your progress through the lens.

Follow the directions of your contact lens fitter / eye care professional.

Cleaning

Each time after using the device clean it with 70% isopropyl alcohol and allow to air dry. Store the device in its case after dry. Plan to replace it every six months.



Precaution: If you apply the device accidentally to the surface of the eye, merely withdraw it from the eye, as there is no suction. Avoid sliding the device over the surface of the eye.



Caution: Please keep this product away from liquids such as hand cream, hair gels, nail polish remover and strong oxidizing agents.



Caution: Choking hazard. Keep this and all other medical devices away from small children.

This product is not recommended to be used with diseased or injured eyes. If you experience an irritated eye, unusual visual impairment or pain, consult your eye care professional.



Caution: Though there is no evidence of allergy issues with this product, **it does contain dry natural rubber. It does not contain liquid latex, a known allergen.**



DO NOT USE this device with hard, gas permeable, scleral or hybrid contact lenses.

If any serious incident has occurred to you in relation to this device, report that to the manufacturer. If you reside in the EU, report the incident to the competent authority of the Member State in which you are established.

This product enables contact lens wearers to insert and/or remove their contact lenses in order to use them according to the intended purpose. Therefore this product is deemed to be a medical device.

Waste Disposal

DMV Corporation recommends to dispose used DMV products according to local regulations in force at the moment of disposal.



This product is a medical device



Protect from direct Sun



Importer



Do not store warmer than 105°C/220°F



Single patient - multiple use



Manufacturer



Read Instructions for Use dmvcorp.com



Date of Manufacturing



Country of manufacture



Authorized Representative



EU
Bilosa Handels GmbH
Haller Strasse 109
6020 Innsbruck, Austria



Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



Switzerland
Prolens AG, Josefstrasse 53
CH-8005, Zürich, Switzerland



MedEnvoy Switzerland
Gotthardstrasse 28
CH-6302 Zug, Switzerland



UK
IGO Optical Ltd
The Granary, Manor Park, Warkworth
Banbury OX17 2AG, UK



Emergo Consulting (UK)
Limited c/o Cr360 – UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ, UK



DMV Corporation
1024 Military Road
Zanesville, Ohio 43701 - USA

